


## 1 Turn the Endostar Provider On: Press the Main Switch.

The standby display will appear.

Standby Display (OTR mode)

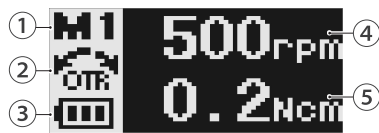

Standby Display (Normal mode)



- ① Memory Number
- ② Rotation Mode
- ③ Battery Power
- ④ Speed Setting
- ⑤ OTR Torque Setting
- ⑥ Torque Reverse Setting

## 2 Select Memory Number: Press the Plus or Minus Switch.

Initial Settings

(These settings can be changed.)


Setting	Memory					
	M1	M2	M3	M4	M5	M6
Speed (rpm)	500	300	500	500	300	500
Reverse Torque (Ncm)	0.2	0.2	0.2	1.0	1.0	1.0
Rotation Mode	OTR	OTR	OTR	Normal (Forward)	Normal (Forward)	Normal (Forward)
Torque Slow Down	—	—	—	Off	Off	Off

## 3 Start Motor: Press the Main Switch again.

The torque display will appear.

\* If you hold the Main Switch down, the motor will run only while it is being held down and stop when you release it.

Torque Display

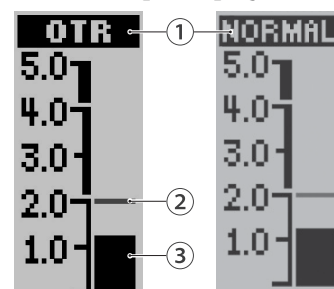

- ① Operation Mode
- ② Current Torque
- ③ Torque Reverse Setting

## 4 Stop Motor: Press the Main Switch again.

The standby display will reappear.

