

endo★star Performer

Endo Motor Instruction Manual

Please read this manual before operating

CE 0197



Contents

1	Product introduction	3
2	Installation	8
3	Function and operation of product	16
4	Operation instruction	21
5	Troubleshooting	39
6	Reprocessing process	40
7	Storage, maintenance and transportation	48
8	Environmental protection	49
9	After service	49
10	European authorized representative	49
11	Symbol instruction	49
12	Statement	51
13	EMC-Declaration of conformity	51

1. Product introduction

1.1. Product description

Endostar Performer endo motor is mainly used in endodontic treatment. It is a cordless endo motor with root canal measurement capability. It can be used as a endo motor for preparation and enlargement of root canals, or device for measuring canal length. It can be used to enlarge the canals while monitoring the position of the file tip inside the canal.

Features:

- Efficient brushless motor, low noise, long service life.
- Cordless portable endo motor with combined length determination.
- 360 degrees rotation of contra angle.
- Adopt real-time feedback technology and dynamic torque control, effectively preventing file separation.


1.2. Model and specification

MotoPex (Endostar Performer)

Please refer to packing list for device configurations.

1.3. Performance and composition

The device is composed of charging base, motor handpiece, contra angle, measuring wire, lip hook, file clip, power adapter, protective silicon cover, etc.

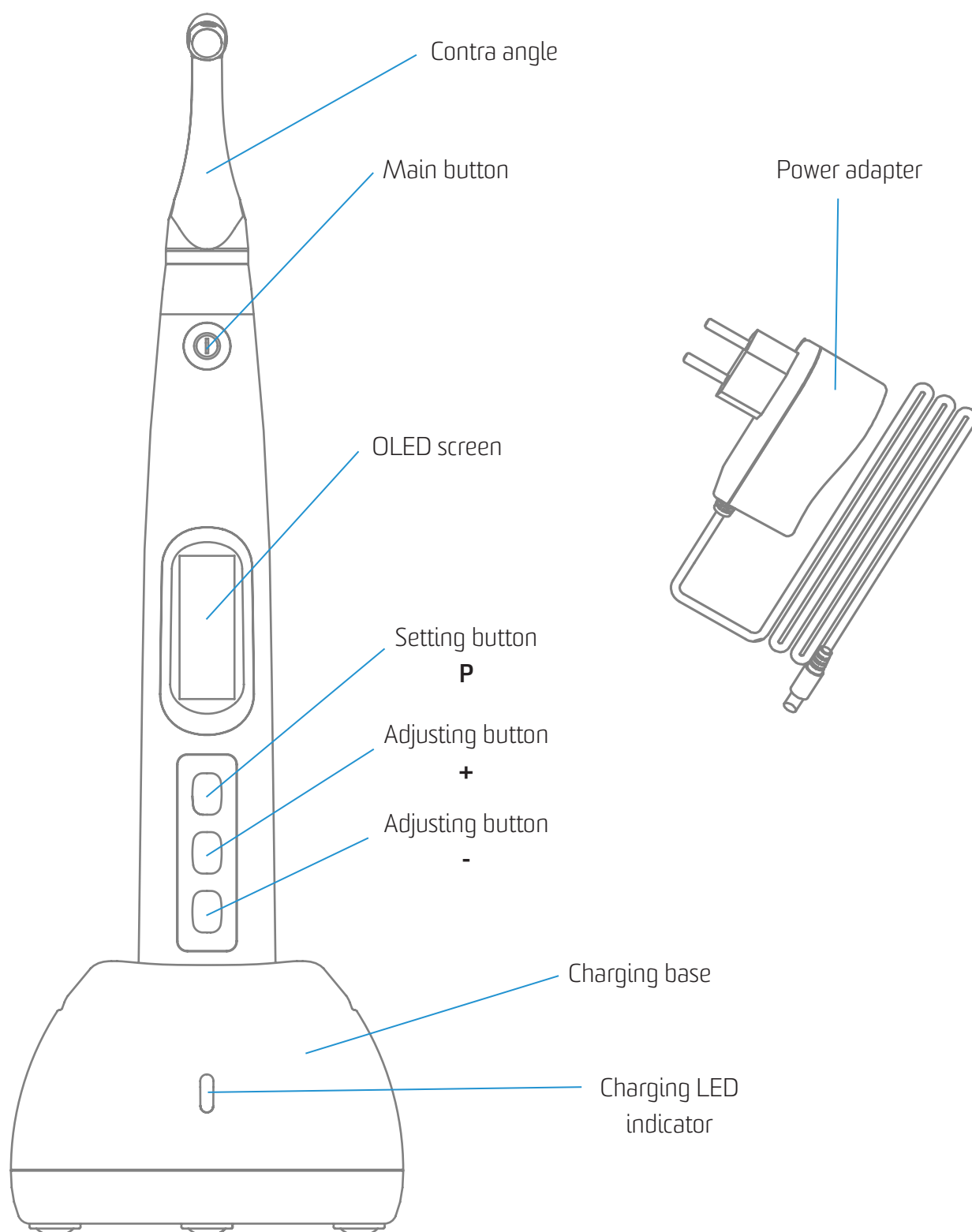

1.4. Indications for Use

Endostar Performer endo motor cordless endodontic treatment motorized handpiece with root canal measurement capability. It can be used for preparation and enlargement of root canals, or measuring the canal length. And it can be used to enlarge the canals while monitoring the position of the file tip inside the canal.

1.5. Scope of application

The device must be operated in hospital and clinic by the qualified dentists.

1.6. Contraindication

- Prohibited for doctor with a pacemaker.
- Do not use this device on patients with implanted pacemakers, defibrillators, or other implantable devices
- Hemophilia patients are banned.
- Use with caution in patients with heart disease, pregnant women and young children.

1.7. Warnings

- Please carefully read this Instruction Manual before first operation.
- This device should be operated by professional and qualified dentist in qualified hospital or clinic.
- Do not directly or indirectly place this device near heat source. Operate and store this device in reliable environment.
- This device requires special precautions regarding electromagnetic compatibility (EMC) and must be in strict accordance with the EMC information for installation and use. Do not use this equipment especially in the vicinity of fluorescent lamps, radio transmitting devices, remote control devices, handheld and mobile high-frequency communication devices.
- Long time use of Reciprocating Motion Mode may result in motor handpiece overheat, thus it should be left to cool for use. If the motor handpiece is overheated frequently, please contact local distributor.
- Please use the original contra angle. Otherwise it will not be used or cause adverse consequences.

- Please do not make any changes to the device. Any changes may violate safety regulations, causing harm to the patient. There will be no promises of any modification.
- Please use original power adapter. Other power adapter will result in damage to lithium battery and control circuit.
- The motor handpiece cannot be autoclaved. Use disinfectant of neutral pH value or ethyl alcohol to wipe its surface.
- Before the contra angle stopping rotating, do not press the push cover of contra angle. Otherwise the contra angle will be broken.
- Before the motor handpiece stopping rotating, do not remove the contra angle. Otherwise the contra angle and the gear inside motor handpiece will be broken.
- Please confirm whether the file is well installed and locked before starting the motor handpiece.
- Please set torque and speed as per the recommended specifications of file manufacturer.
- Error in replacing lithium batteries can lead to unacceptable risks, so use the original lithium battery and replace the lithium battery according to the correct steps in the instructions.
- Do not place the equipment where it is difficult to disconnect the power supply.
- Please remove the battery if the motor handpiece is not likely to be used for some time.

1.8. Device safety classification

- Type of operation mode: Continuous operating device.
- Type of protection against electric shock: Class II equipment with internal power supply.
- Degree of protection against electric shock: B type applied part.
- Degree of protection against harmful ingress of water: Ordinary equipment (IPX0).
- Degree of safety application in the presence of a flammable anesthetic mixture with air, oxygen, or nitrous oxide: Equipment cannot be used in the presence of a flammable anesthetic mixture with air, oxygen, or nitrous oxide.
- Applied part: contra angle, lip hook, file clip, touch probe.
- The contact duration of applied part: 1 to 10 minutes.
The temperature of the surface of applied part may reach 46.6°C.

1.9. Primary technical specifications


- Battery - Lithium battery in motor handpiece: 3.7V /2000mAh
- Power adapter (Model: ADS-6AM-06N 05050/UE08WCP- 050100SPA), Input: ~100V-240V 50Hz/60Hz 0.4A Max Output: DC5V/1A.
- Torque rang: 0.2Ncm-5.0Ncm (2mNm ~ 50mNm).
- Speed rang: 100rpm~1800rpm.
- Wireless charging: Frequency range: 112-205KHz Maximum RF output power of the product: 9.46dBuA/m@3m.

1.10. Environment parameters

- Environment temperature: +5°C ~ +40°C.
- Relative humidity: 30% ~ 75%.
- Atmospheric pressure: 70kPa ~ 106kPa.

2. Installation


2.1. Basic accessories of product

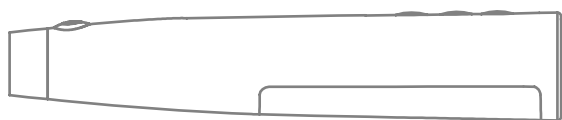


Motor handpiece




Contra angle

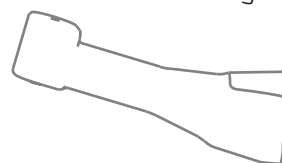


Protective silicon cover

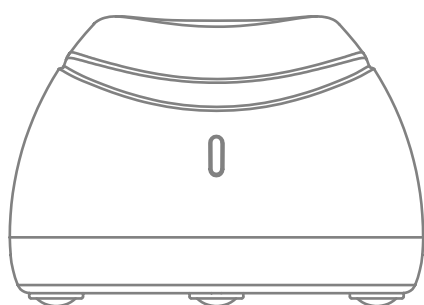


Charging base

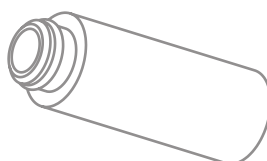


Nozzle

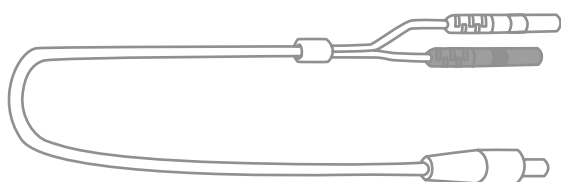


Measuring wire

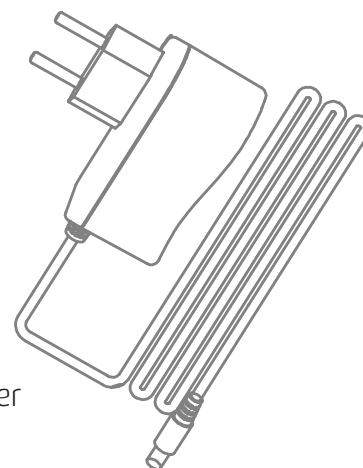

Power adapter

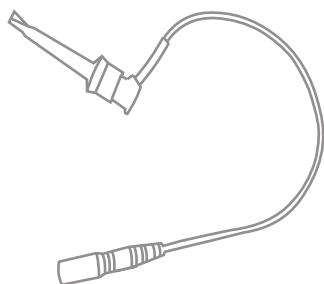

File clip

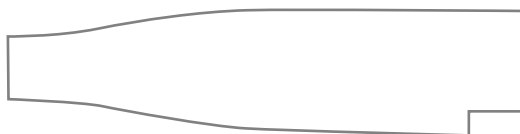

Disposable insulation sleeves



Touch probe




Lip hook

2.2. Display Screens

Display Screens for 5 Operation Modes and Standby.

- EAL Mode.

This mode is for canal measurement. The motor handpiece does not run in this mode.

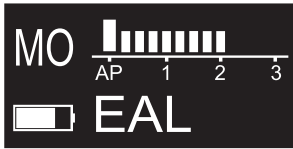


- CW Mode.

The motor handpiece rotates forward 360°, clockwise direction.



- CCW Mode.

The motor handpiece rotates counterclockwise direction only. This mode is used to inject calcium hydroxide and other medicant. When this mode is being used, a double-beep sounds continuously.

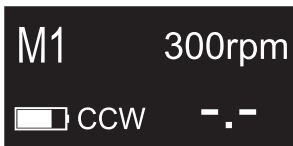

- F-R Mode.


Forward-reverse motion (CW-CCW). Safe canal preparation through alternating, partial rotations of the file forward (F) and reverse (R). Set the range of each file rotation by entering the angles for F and R. A full rotation is 360 degrees.



It is suggested that the difference between the forward angle and reverse angle should be greater than or equal to 120 degrees, otherwise, root canals cannot be prepared effectively. Forward Angle < Reverse Angle, such as F:60/R:150, effective cutting angle is Reverse Angle, it is suitable for used the reciprocating files.

- ATR Mode.


ATR: Adaptive Torque Reverse function.

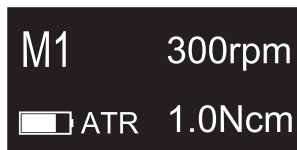

Normal continuous forward rotation, when the load of the file is greater than the set torque limit, the file will start to rotate alternately at the set angle.


- Torque Display.

This appears when the motor is running. Meter shows the torque load on the file.

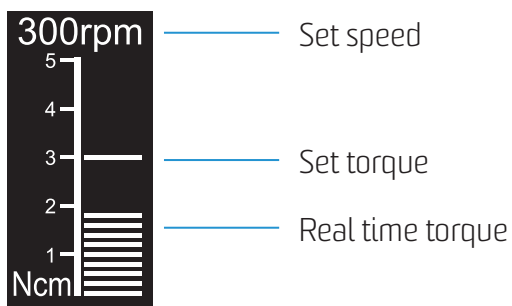

- Canal Measurement Display.

This appears when a file is inside the canal and the lip hook is contacting the patient mouth. Bars in meter show the location of the file tip. In the EAL Mode, if the length is less than 1.0, the display will be enlarged.

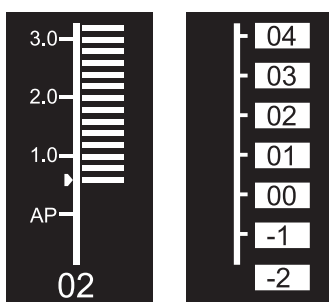

The meter numbers 1.0, 2.0, 3.0 and digital numbers 00-16 do not represent the actual length from the apical foramen. It simply indicates the file progression towards the apex. The digital numbers -1 and -2 indicate that the file has passed the apex foramen. The digital number "00" indicate that

the file has reached the apex foramen. Subtract 0.5-1mm from the measured file length as the working length. These numbers are used to estimate the canal's working length.


2.3. Instructions for contra angle

- The contra angle adopts precision gear transmission, and the transmission ratio is 6:1.
- Before the first use and after treatments, please clean and disinfect contra angle with disinfectant of neutral PH value. After disinfection, lubricate it with specific cleaning oil. Finally, sterilize it under high temperature and high pressure (134°C, 2.0bar~2.3bar (0.20MPa~0.23MPa)).
- The contra angle can only be used cooperatively with this device. Otherwise the contra angle will be damaged.

2.4. Installation and removal of contra angle

- Installation.


Align any locating pin of the contra-angle with the positioning slot on the motor handpiece and push the contra-angle horizontally. The three locating pins on the contra-angle are inserted into those three positioning holes on the motor handpiece. A "click" sound indicates that the installation is in place. The contra-angle can be rotated 360° freely.

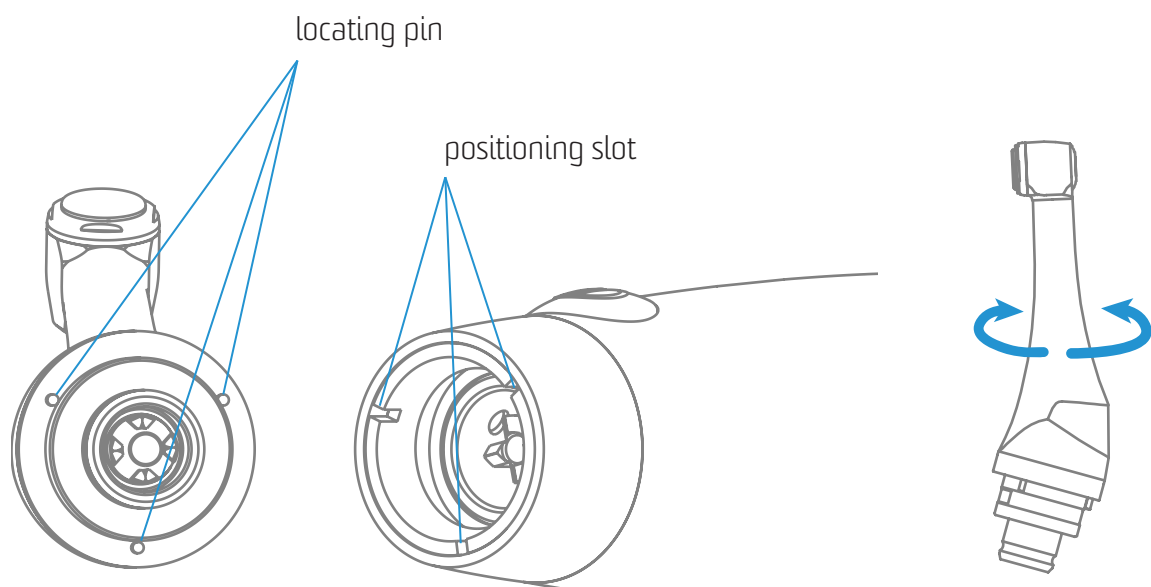

The contra-angle is free to rotate, adapting to the root canal of different positions, and it is convenient to watch the screen when operating.

Removal.

Pull out the contra angle horizontally when the motor handpiece does not run.

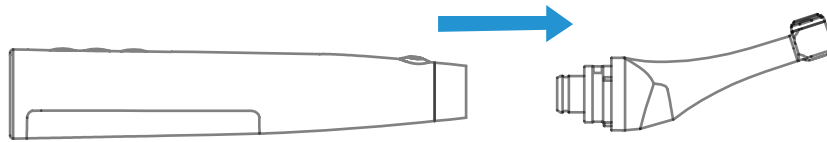


Warnings

Before plugging in or pulling out contra angle, please first stop the motor handpiece. After installation, please check and confirm that the contra angle has been well installed.

2.5. Installation and removal of file

Installation of file.

Before starting the device, plug the file into the hole of contra angle head. Hold down the push button on the contra angle and insert the file. Turn the file back and forth until it is lined up with interior latch groove and slips into place. Release the button to lock the file into the contra angle.

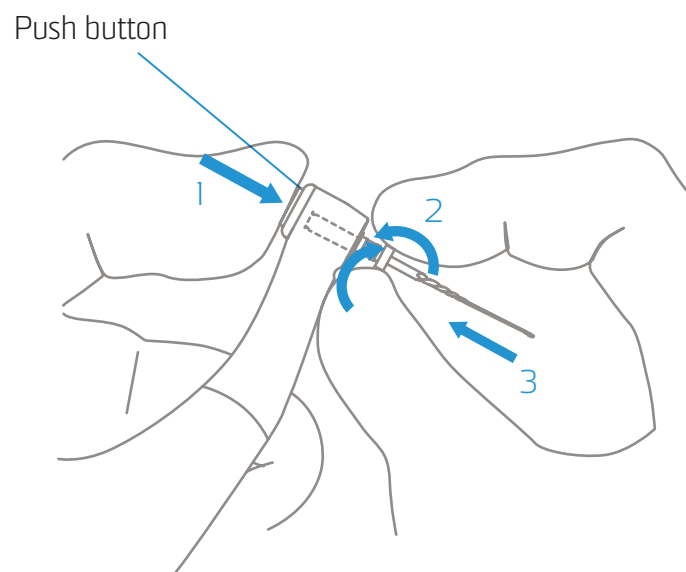


Warnings

After plugging the file into contra angle, let go the hand on push cover to assure that the file cannot be taken out. Be careful when inserting files to avoid injury to fingers. Inserting and removing files without holding the push button may damage the chuck of contra angle. Please use files with shanks meet the ISO standard. (ISO standard: $\varnothing 2.334 - 2.350$ mm).

- Removal of file.

Pressing the push cover, and then directly pull out the file.

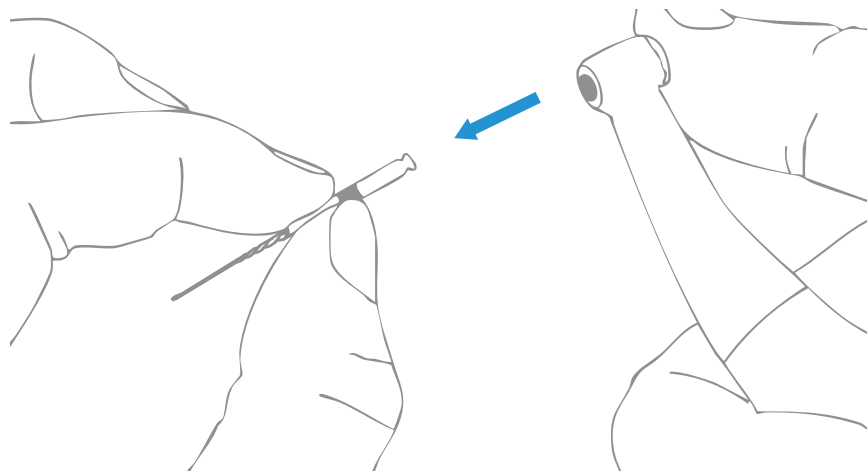


Warnings

Before plugging and pulling out the file, the motor handpiece must be stopped. Be careful when removing files to avoid injury to fingers. Removing files without holding the push button will damage the chuck of contra angle.

2.6. Canal measurement functional connection.

This is not required if the canal measurement function will not be used. Connect the measuring wire to the motor handpiece. Line up the measuring wire plug with the notch on the back of the motor and push it all the way in. Connect the file clip plug into the socket (black) on the measuring wire. Connect the lip hook to the socket (white) on the measuring wire.

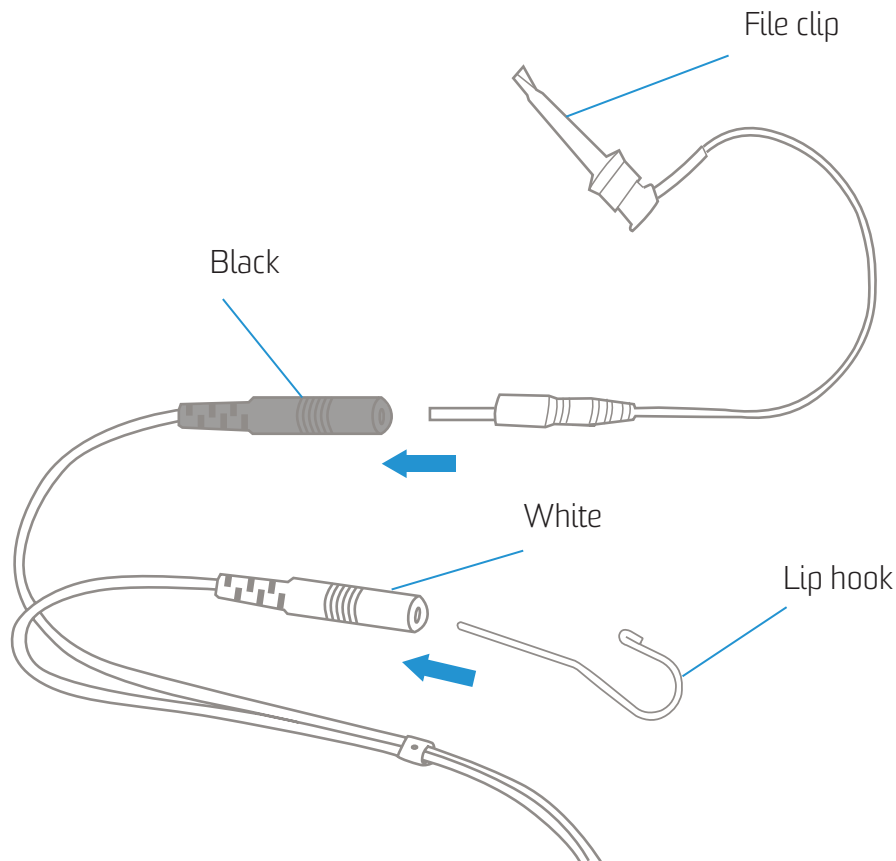

Warnings

Connect the lip hook to the socket (white) on the measuring wire. Otherwise, the function of root canal preparation and root canal length measurement cannot be used together.

2.7. Installation and removal of disposable insulation sleeves.

- Installation.

Before each use of the handpiece and after the handpiece is cleaned and disinfected, put on a disposable isolation sleeve. Take the isolation sleeve out of the isolation sleeve box, then insert the isolation sleeve into the motor handpiece from the thin end of the handpiece, and install the isolation sleeve until there is no obvious wrinkle. After installing the disposable isolation sleeve, wrap the barrier film around the handpiece surface. After that, clean and disinfect the surface of the handpiece. Refer to Chapter 6.3 for cleaning and disinfection procedures.

- Removing.


After each use, remove the barrier film and slowly pull the isolation sleeve from the thin end of the handpiece.

Warnings

Isolation sleeves are not reusable .

3. Function and operation of product

3.1. Button definition and settings

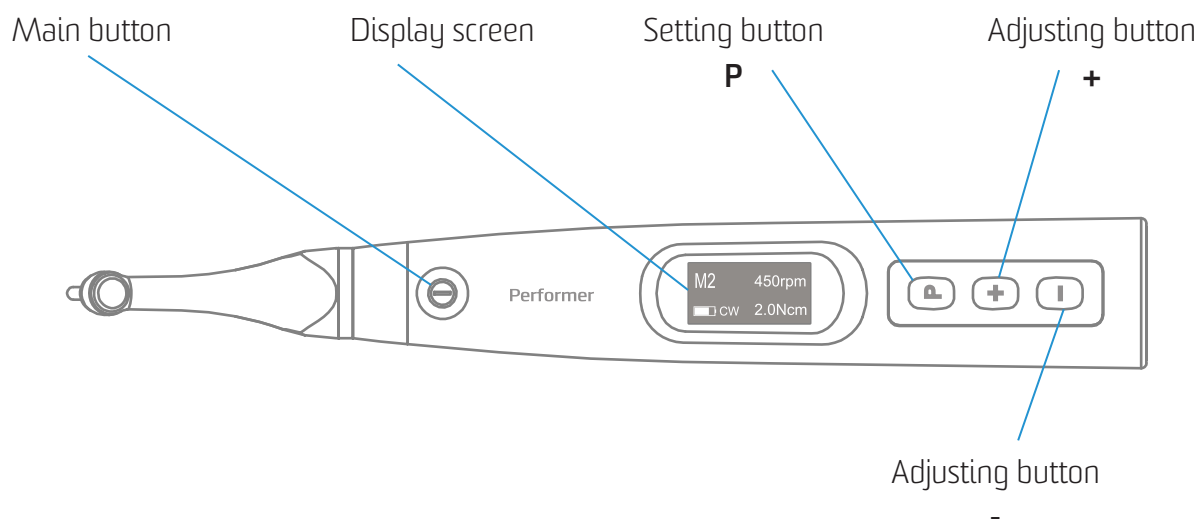

- Turn power on.

Press Main button to turn on motor handpiece.

- Turn power off.

Hold down the Setting button "P", then press Main button to turn off motor handpiece.

- Customized program change.

Press Adjusting button "+" / "-" during standby state.

- Parameter setting.

Press Setting button "P" till target parameters, press Adjusting button "+" / "-" to change, then press Main button or wait 5 seconds to confirm.

- Preset program selection.


Long press Setting button "P" to entry preset program during standby state, press Adjusting button "+" / "-" to select file system, press Setting button "P" to entry select file number, press Adjusting button "+" / "-" to select file number, then press Main button to confirm.

- Handpiece functions setting.


With the motor handpiece turned off, hold down the Setting button "P" and press Main button to entry handpiece functions setting, press Setting button "P" till target setting, press Adjusting button "+" / "-" to adjust, then press Main button to confirm.

3.2. Screen display


- Standby interface

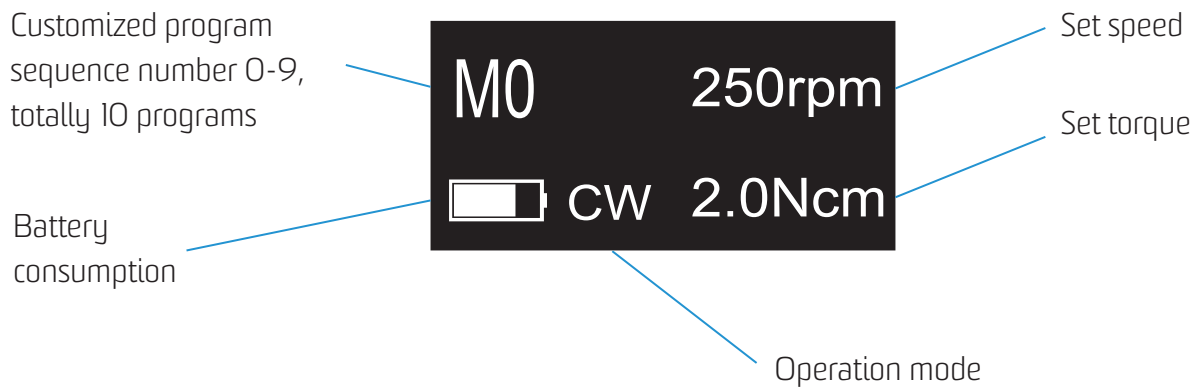


- Working interface

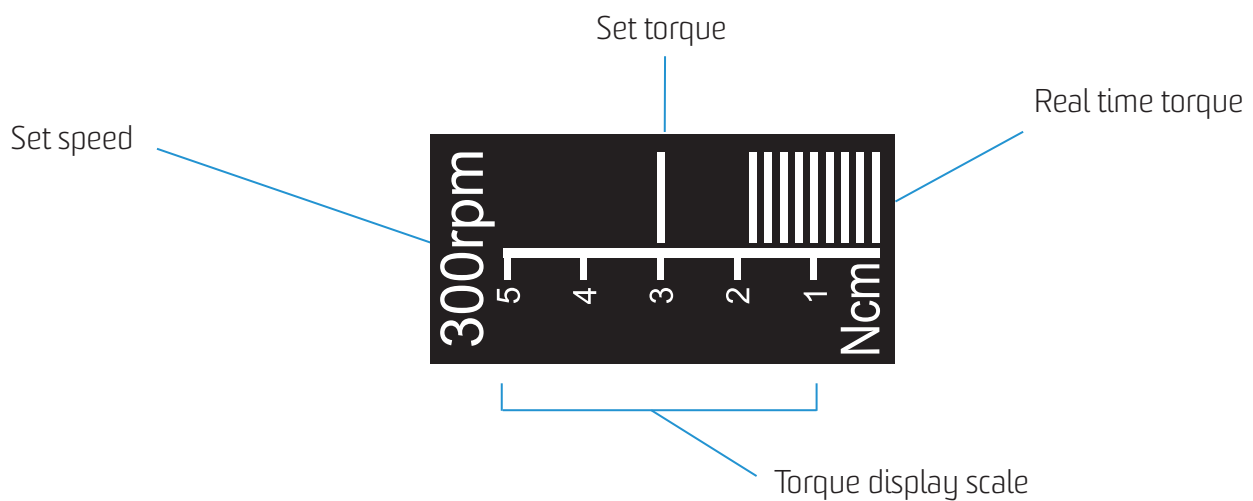

- Canal measurement mode interface

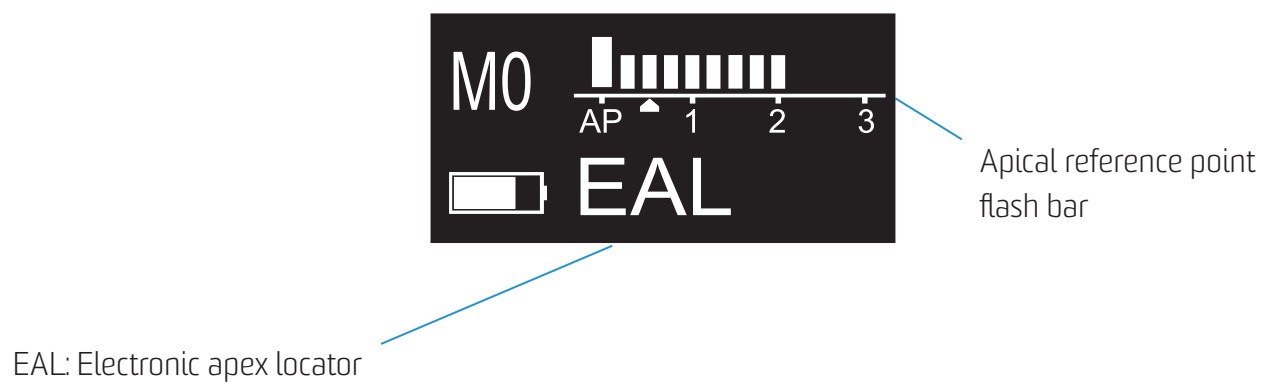


- Canal measurement state interface

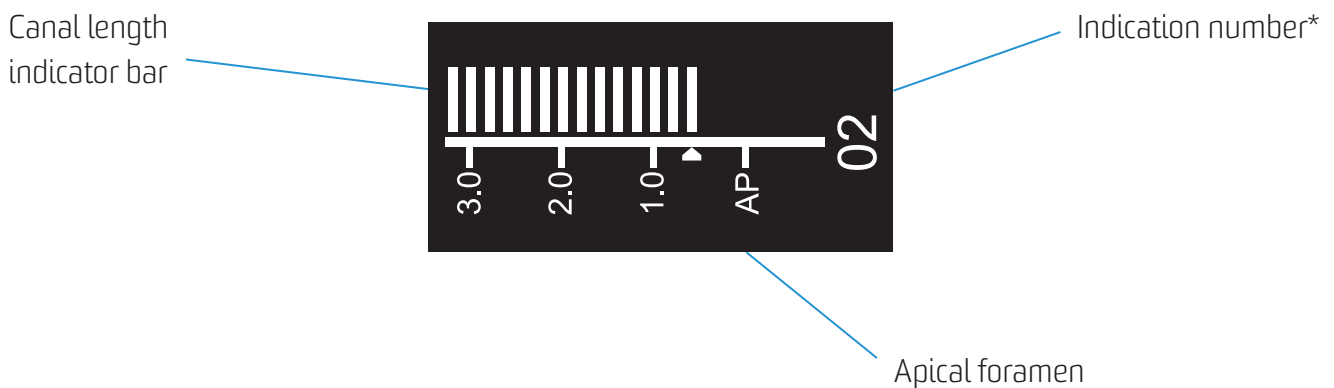

*Digital numbers 00-16 do not represent the actual length from the apical foramen. It simply indicates the file progression towards the apex. Number "00" indicate that the file has reached the apical foramen.

- Apical reference point setting interface

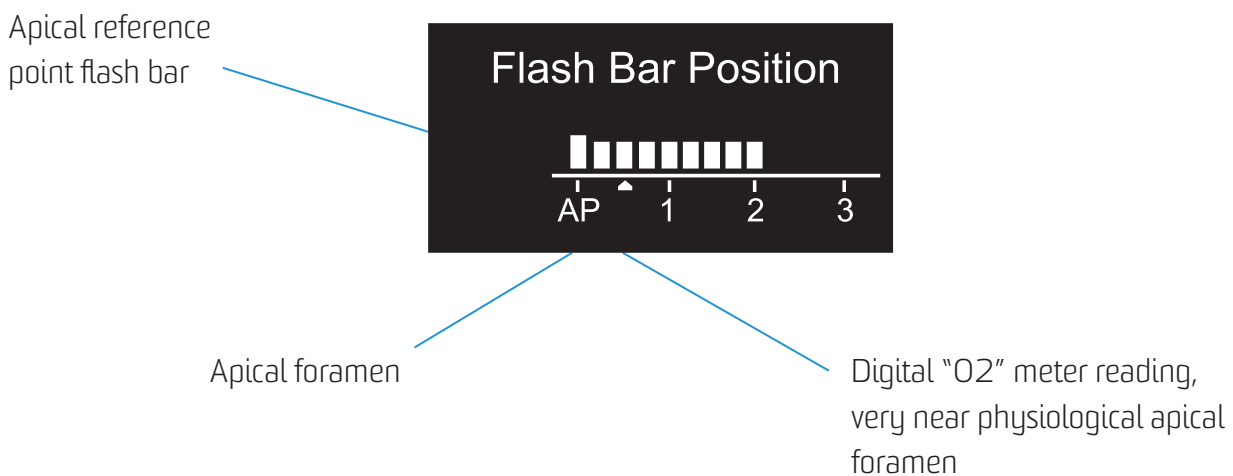

3.3. Terms and definition

- CW

Clockwise rotation, forward ration. Be applied to rotary file.

- CCW

Counter clockwise rotation, reverse rotation. Be applied to special file, inject calcium hydroxide and other solutions.

- F-R

Forward-reverse motion (CW-CCW). Safe canal preparation through alternating, partial rotations of the file forward (F) and reverse (R). Set the range of each file rotation by entering the angles for F and R. A full rotation is 360 degrees. It is suitable for using the reciprocating files.

- ATR

Adaptive torque reverse. Up to setting torque, the motor will move with ATR mode; when torque reduces to normal value, the motor will clockwise rotate.

- Forward Angle

Angle of clockwise rotation of the file.

- Reverse Angle

Angle of counter clockwise rotation of the file.

- EAL

Electronic apex locator. In the mode, the device will work like a stand-alone apex.

- AP

Apical foramen.

- Apical Action

The file action when file tip reaches the flash bar point.

- Flash Bar Position

Shows the point inside the canal where specified apical action is triggered.

- Auto Start

The file rotation starts automatically when the file is inserted in the canal.

- Auto Stop

The file rotation stops automatically when the file is taken out of the canal.

- Apical Slow Down

The file slows down automatically as it approaches the apex. Activating in CW and CCW operation mode.

- Operation Mode

5 operation modes for canal shaping and measurement. Such as CW, CCW, F-R, ATR and EAL.

- Speed

File rotation speed.

- Torque (Torque Limit / Trigger Torque)


For CW and CCW modes, the torque value (Torque Limit) that triggers reverse rotation. For ATR mode, the torque value (Trigger Torque) that triggers ATR action.

4. Operation instruction


4.1. Power on and power off

- Starting and stopping of motor handpiece.

Under the power off state of motor handpiece, press Main button, and then the motor handpiece will enter Standby interface. The interface displays are as follow:

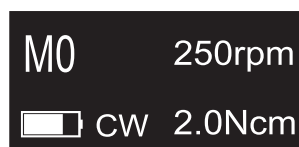

Under Standby interface, press Main button, and then the motor handpiece will enter Working interface. The interface displays are as follow:

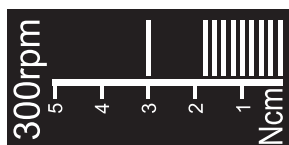


Press the Main button again, and then the motor handpiece backs to Standby interface. Hold down the Setting button "P", then press Main button to turn off motor handpiece. In Standby Interface, the motor handpiece would automatically shut down after 3 minutes without any button-pressing operation. The motor handpiece will also automatically shut down while it is put into the charging base.

4.2. Selecting customized program sequence number


The motor handpiece has 10 memory programs(M0-M9) and 5 preset programs, press Adjusting button "+" / "-" to change customized program sequence number during standby state. M0-M9 is a memory program for canal shaping and measurement, every memory program has its own parameters such as Operation mode, speed and torque, all these parameters can be changed.

4.3. Parameter setting


- Before starting of motor handpiece, please check the operation mode is correct. All the parameters must be set according to files, make sure all the parameters are expected before starting of motor handpiece, otherwise has risk of file separate.

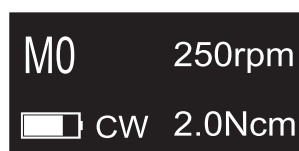

- It has 5 operation modes for canal shaping and measurement: CW, CCW, F-R, ATR and EAL (see chapter 3.3 Terms and definition to get the explanations of these modes). Press Setting button "P" once during standby state, press Adjusting button "+"/"-" to select correct Operation mode. CCW mode is used to inject calcium hydroxide and other medicant. When this mode is being used, a double-beep sounds continuously, used for indicating counter clockwise rotation happening.

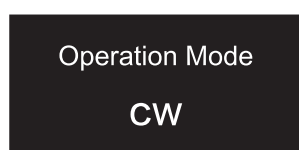

- Repeatedly press Setting button "P" to check all the next level parameters of this operation mode are expected, press Adjusting button "+"/"-" to select if not.
- The speed setting can be adjusted from 100 rpm to 1800 rpm. Press Adjusting button "+"/"-" to increase or decrease speed. Long press to fast increase or fast decrease speed. In ATR mode, speed of 100~500rpm are available. In F-R mode, speed of 100~500rpm are available.

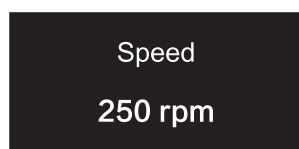

- The torque setting can be adjusted from 0.2Ncm to 5.0Ncm. Press Adjusting button "+"/"-" to increase or decrease torque. Long press to fast increase or fast decrease torque. In ATR mode, the Trigger Torque of 0.2Ncm~ 4.0Ncm are available.

Torque Limit

2.0 Ncm

- Actions that happen automatically when the file tip reaches the point inside the canal determined by the Flash Bar setting. Benefit from integration of length determination, when the file reaches the reference point, the motor will response according to setting, it can be Reverse , Stop and OFF. Press Adjusting button "+" / "-" to change. OFF: Disable Apical Action function, file rotating as usual even if reach the reference point. Stop: automatically rotation stop when reach the reference point, upward a little bit and will rotate again. Reverse: automatically reverses rotation when reach or pass the reference point, upward a little bit, the rotation direction will change back again.

Apical Action

OFF

- Rotation starts automatically when the file is inserted into the canal and the canal length indicator bar lights up more than 2 bars. Press Adjusting button "+" / "-" to change. OFF: Motor does not start when file is inserted into the canal. The Main button is used to start and stop the motor handpiece. ON: Motor starts automatically.

Auto Start


OFF

- Rotation stops automatically when the file is taken out of the canal and the canal length indicator bar lights up less than 2 bars before the file is taken out. Press Adjusting button "+" / "-" to change. OFF: Motor does not stop when file is taken out the canal. The Main button is used to start and stop the motor handpiece. ON: Motor stops automatically.

Auto Stop
OFF

- This is the reference point where various apical actions are triggered. Press Adjusting button "+" / "-" to select reference point by change the flash bar. The meter's 0.5 reading indicates that the file tip is located very near the physiological apical foramen. The reference point (flash bar) can be set from 2 to AP (Apex) on the meter.

Flash Bar Position

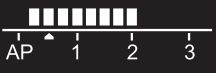

The diagram shows a horizontal scale with labels AP, 1, 2, and 3. Above the scale, there are eight vertical bars of increasing height from left to right. The first bar is at the AP position, and the eighth bar is at the 2 position. A small upward-pointing arrow is located between the AP and 1 labels, indicating the current flash bar position at 0.5.

- Rotation automatically slows down as the file tip approaches the reference point. Press Adjusting button "+" / "-" to change. OFF: Disable Apical Slow Down function. ON: Rotation automatically slows down as the file tip approaches the reference point.

Apical Slow Down
OFF

- Forward Angle. In the F-R mode, the Forward Angle of 20°~400° are available. In the ATR mode, the Forward Angle of 60°~400° are available.
- Reverse Angle. In the F-R mode, the Reverse Angle of 20°~400° are available. In the ATR mode, the reverse Angle cannot be greater than the forward Angle.

Forward Angle
30°

Reverse Angle
30°

M1 F:30°
F-R R:30°


4.4. The most common Endostar file systems are already pre-set.

- To select a pre-set Endostar file system from the standby state, press and the 'P' setting button.
- To select the Endostar file system, press the '+'/'-' adjustment button and press the settings button 'P' to confirm.
- The file settings can be changed manually.
- List of Endostar file systems and their settings.


	CW		ATR		F-R
E3 Azure Small	1.5 Ncm	300 rpm	0.2 Ncm	300 rpm	2,5 Ncm, 300 rpm, forward Angle 150°, Reverse Angle 60°
E3 Azure Basic	2.00 Ncm	300 rpm			
E3 Azure Big	2.00 Ncm	300 rpm			
EP Easy Path	1.5 Ncm	300 rpm	0.2 Ncm	300 rpm	2,5 Ncm, 300 rpm, forward Angle 150°, Reverse Angle 60°
REvision	2.0 Ncm	500 rpm	0.2 Ncm	300 rpm	2,5 Ncm, 300 rpm, forward Angle 150°, Reverse Angle 60°
RE Endo Rotary System	2.0 Ncm	500 rpm	0.2 Ncm	300 rpm	2,5 Ncm, 300 rpm, forward Angle 150°, Reverse Angle 60°

4.5. Preset program selection


- For convenience, we preset some common file system. Press Adjusting button "+"/"-" to switch to preset program (M0-M9, preset program 1-5), the interface will show as left.

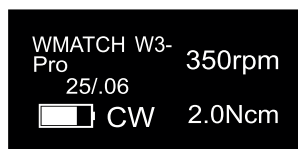


- Long press Setting button "P" to entry preset program during standby state, the interface will show as left. Press Adjusting button "+" / "-" to select file system.

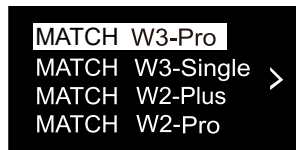

- After select file system, press Setting button "P" to entry select file number, press Adjusting button "+" / "-" to select file number, then press Main button to confirm.

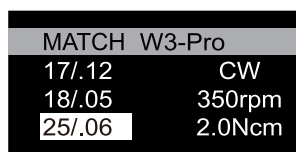

- The parameters of "W3-Pro" can also be changed make it different from default setting. If want to change back to default setting, long press Setting button "P" to entry preset program during standby state, select "W3-Pro" and press "Main" button to confirm, the default setting will be reloaded. Turn off the motor handpiece and then power on, the preset program can also restore the default setting. Changing the preset program default setting is not recommended, otherwise has risk of file separate.

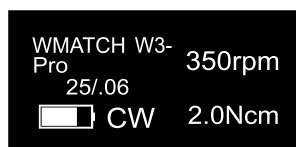

4.6. Handpiece functions setting

With the motor handpiece turned off, hold down the Setting button "P" and press Main button to entry handpiece functions setting, press Setting button "P" till target setting, press Adjusting button "+" / "-" to adjust, then press Main button to confirm.

- With the motor handpiece turned off, hold down the Setting button "P" and press Main button to entry handpiece functions setting, the software version number will appear on the display screen.

Software Version

V1.0.0

- After 3 seconds of displaying the version number on the screen, the "Auto Power OFF" can be change, press Adjusting button "+" / "-" to adjust, then press to "Main" button to confirm. No buttons are pressed, auto power off time of motor handpiece. It can be set from 3 to 30 minutes in 1 minute increments.

Auto Power OFF

5 min

- Press Setting button "P" again, the "Auto Standby Scr" can be change, press Adjusting button "+" / "-" to adjust, then press to "Main" button to confirm. No buttons are pressed, auto return to standby display of motor handpiece. It can be set from 3 to 30 seconds in 1 second increments.

Auto Standby Scr


30 sec

- Press Setting button "P" again, the "Dominant Hand" can be change, press Adjusting button "+" / "-" to adjust, then press to "Main" button to confirm. The right hand and the left hand can be set.


Dominant Hand

Right


- Press Setting button "P" again, the "Calibration" can be change, press Adjusting button "+" / "-" to select "ON", then press to "Main" button to calibration. Before calibrating, making sure the original contra angle is installed, and do not install the file. The torque will not correct if calibration without original contra angle or any load on contra angle chuck, and has risk of file separate. After replacement of contra angle, the contra angle shall be calibrated before use.

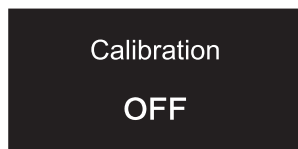

- Press Setting button "P" again, the "Beeper Volume" can be change, press Adjusting button "+" / "-" to adjust, then press to "Main" button to confirm. The "Beeper Volume" can be set from 0-3. Vol.0: Mute.

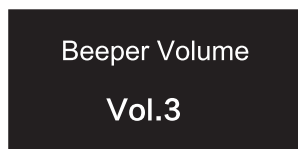


- Press Setting button "P" again, the "Restore Defaults" can be change, press Adjusting button "+" / "-" to select "ON", then press to "Main" button to restore defaults.

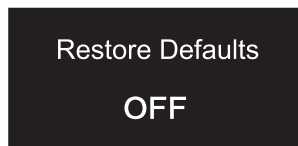

4.7. Protective function of automatic reverse

During operation, if the load value exceeds the preset torque value, the file rotation mode will automatically change to Reverse Mode. And the file would return to normal rotation mode when the load is below the preset torque value again.

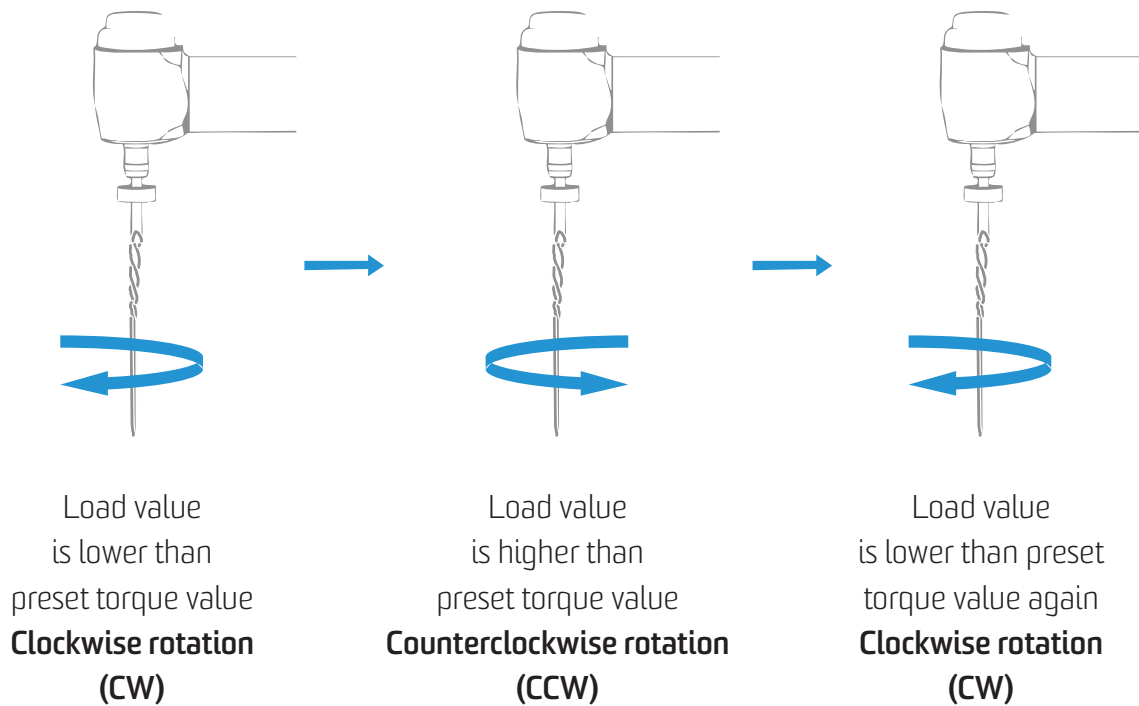

Warnings



- Protective function of automatic reverse is ONLY suitable for CW mode.
- This function is forbidden under CCW mode, ATR mode.
- When the motor handpiece battery indicator indicates a low battery capacity, the low battery capacity is insufficient to support the motor handpiece to reach the limit torque value, that is, the auto-reverse function will not work properly. Please charge it in time.
- If the motor handpiece is under load all the time, the machine may stop automatically as a result of overheat protection. If it happens, turn off the motor handpiece for a while until the temperature drops.

4.8. Motor operation

Please set operation mode, torque and speed as per the recommended specifications of file manufacturer.



- Motor alone mode

When using as motor alone mode, the torque bar will show on the screen (more information about torque bar, please see chapter 3. 2 Screen display).

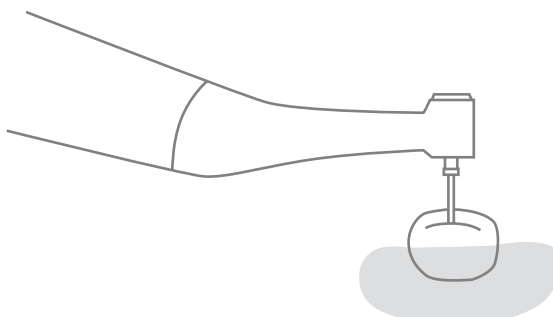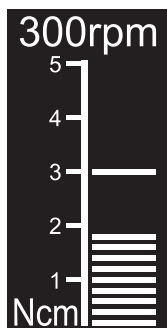



- Motor combined canal measurement function mode

When using motor combined canal measurement function, the measuring wire must be connecting with motor handpiece by USB socket, and white socket connects with patient's lip by lip hook, keep the black socket idle. The canal length indicator bar will show on the screen (more information about canal length indicator bar, please see chapter 3. 2 Screen display). Setting parameters of automatic functions as needed, such as Apical Action, Auto Start, etc (more information about automatic functions, please see chapter 4.3 Parameter setting).

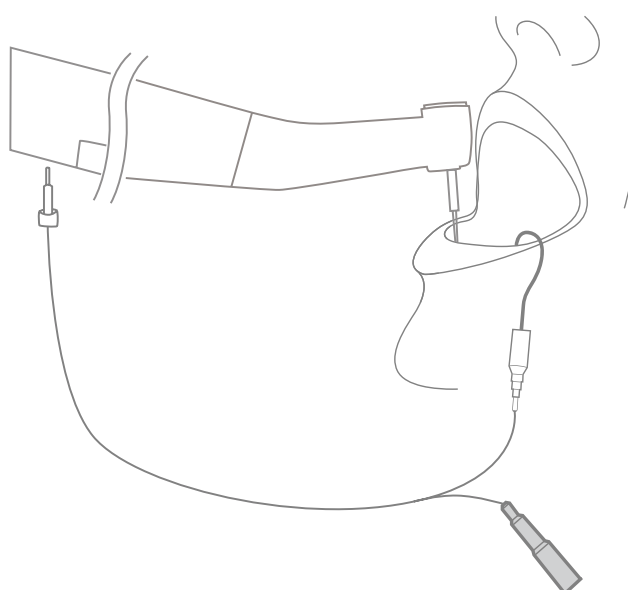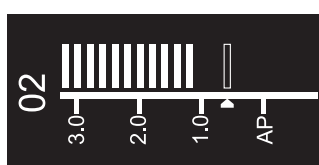


- Connection testing

Strongly recommend check the connection testing every time before use. Touch the lip hook with the file in the contra angle and check that all the bars on the meter on the screen light up, and the motor should be reversed continuously, otherwise, the measuring wire or contra angle should be replaced.

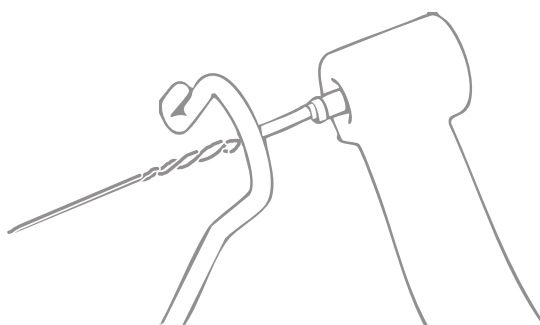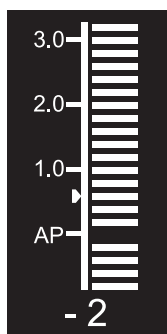


4.9. Canal measurement operation

When using as alone apex locator mode. We suggest put the motor handpiece on the charging base to get better visual angle. Press Setting button "P" once during standby state, press Adjusting button "+" / "-" to select EAL Operation mode, then press Main button to confirm (see chapter 3.3 Terms and definition to get the explanations of Operation modes). The measuring wire must be connecting with motor handpiece by USB socket, white socket connects with patient's lip by lip hook, and black socket connect with file clip. The canal length indicator bar will show on the screen (more information about canal length indicator bar, please see chapter 3. 2 Screen display).

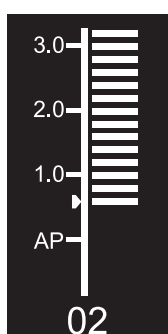





The file clip must hold the file correctly. Push the button on the file clip with your thumb in the direction shown by the arrow. Clip the holder onto the metal upper part of the file and then release the button.

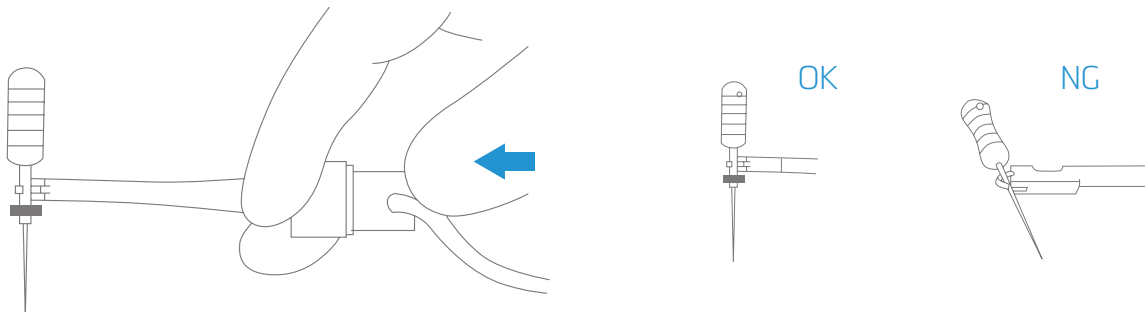

- Connection testing

Strongly recommend check the connection testing every time before use. Clip the holder onto lip hook and check that all the bars on the meter on the screen light up, otherwise, the measuring wire or file clip should be replace.

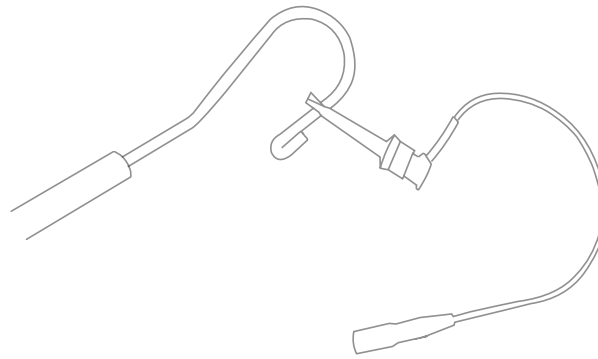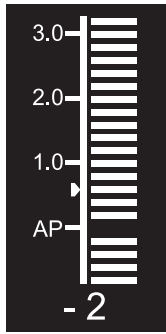


Root canals not suitable for canal measurement

Accurate measurement cannot be obtained if the root canal conditions shown below.

- Root canal with a large apical foramen.

Root canal that has an exceptionally large apical foramen due to a lesion or incomplete development cannot be accurately measured. The results may show shorter measurement than the actual length.

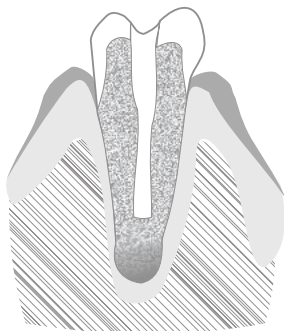


- Root canal with blood overflowing from the opening

If blood overflows from the opening of the root canal and contacts the gums, this will result in electrical leakage and an accurate measurement cannot be obtained. Wait for bleeding to stop completely. Clean the inside and opening of the canal thoroughly to get rid of all blood, and then make a measurement.


- Root canal with a chemical solution overflowing from the opening.

An accurate measurement cannot be obtained if some chemical solution is overflowing from the canal opening. In this case, clean the canal and its opening. It is important to get rid of any solution overflowing the opening.

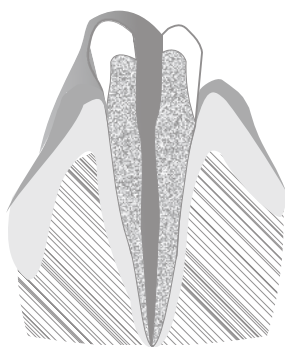


- Broken crown

If the crown is broken and a section of the gingival tissue intrudes into the cavity surrounding the canal opening, contact between the gingival tissue and the file will result in electrical leakage and an accurate measurement cannot be obtained. In this case, build up the tooth with a suitable material to insulate the gingival tissue.

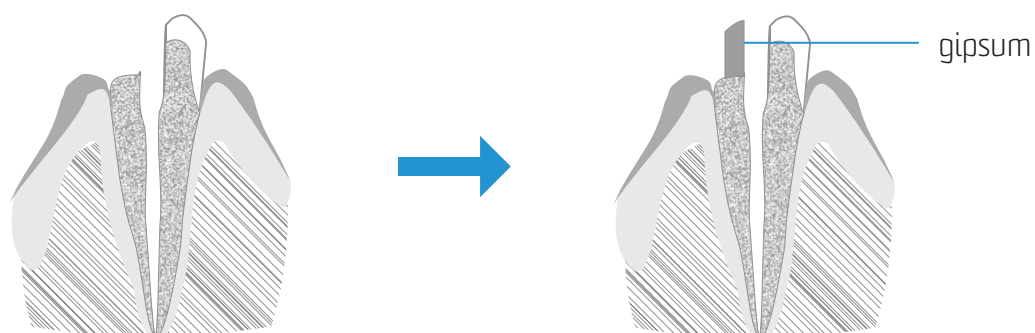


- Fractured tooth

Leakage through a branch canal Fractured tooth will cause electrical leakage and an accurate measurement cannot be obtained. A branch canal will also cause electrical leakage.

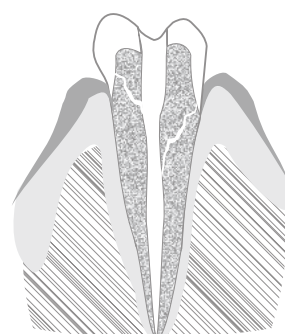


- Re-treatment of a root filled with gutta-percha

The gutta-percha must be completely removed to eliminate its insulating effect. After removing the gutta-percha, pass a small file all the way through the apical foramen and then put a little saline in the canal, but do not let it overflow the canal opening.

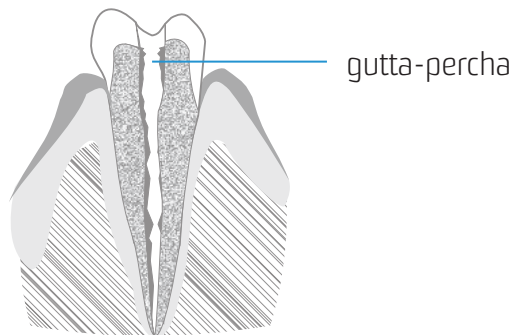


- Crown or metal prosthesis touching gingival tissue

Accurate measurement cannot be obtained if the file touches a metal prosthesis that is touching gingival tissue. In this case, widen the opening at the top of the crown so that the file will not touch the metal prosthesis before taking a measurement.

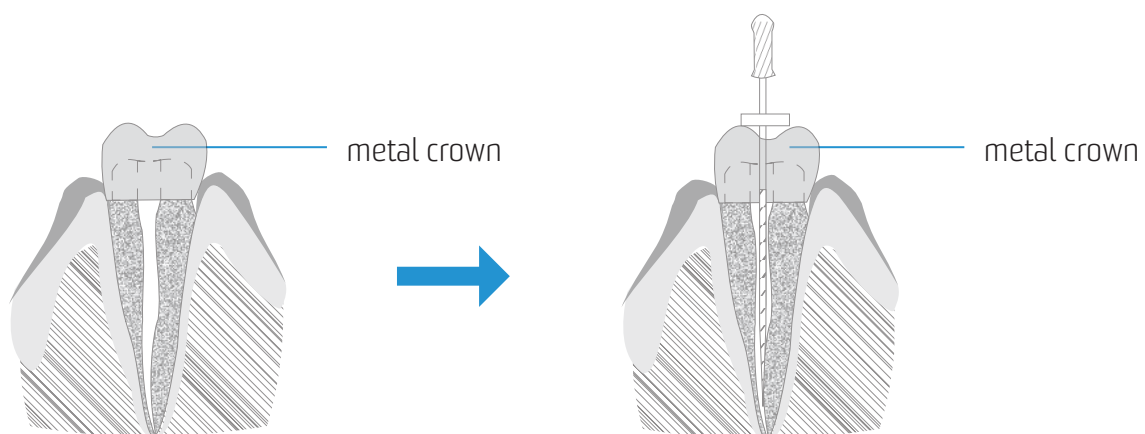

- Extremely dry canal

If the canal is extremely dry, the meter may not move until it is quite close to the apex. In this case, try moistening the canal with saline.

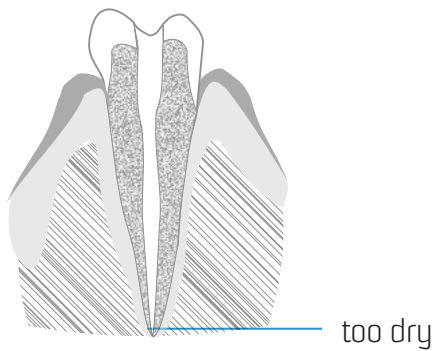


- Difference measuring result between apex locator reading and radiography

Sometimes the reading of apex locator and the X-ray image will not correspond. This does not mean that the apex locator is not working properly or that the X-ray exposure is a failure. An X-ray image might not show the apex correctly depending on the angle of the X-ray beam, and the location of the apex might seem to be other than it really is.

- The actual apex for the canal is not the same as that for the anatomical apex

There are frequently cases where the apical foramen is located up towards the crown. In these cases, an X-ray might indicate that the file has not reached the apex even though it has actually reached the apical foramen.

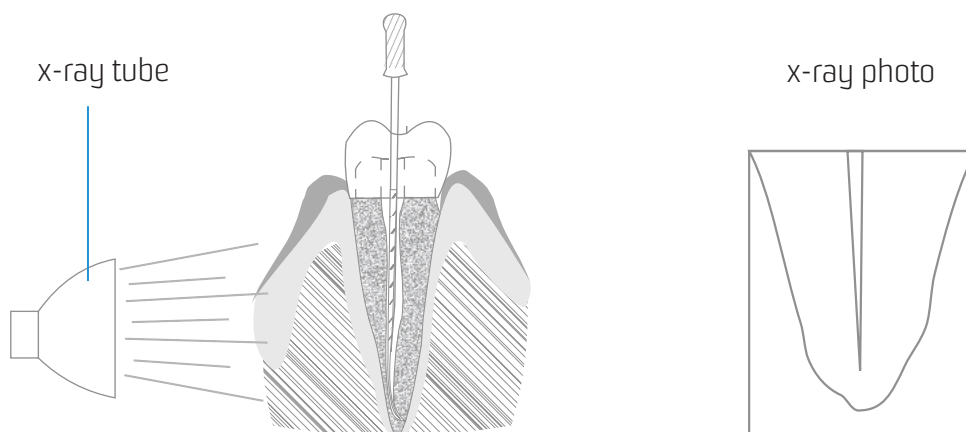

4.10. Battery Charging

The motor handpiece has built-in rechargeable lithium battery. When charging the battery, leave approximately 10cm around the charging base for easy access to inlet and the power cord. Connect

the power adapter with the charging base. Confirm that it is well connected, and then place the motor handpiece into the charging base. If the indicator light on charging base turns blue, it indicates that it is charging. If the indicator light on base turns green, it indicates that the battery capacity is enough, and there is no need to charge. After charging, please unplug the power adapter.

4.11. Replacing Battery

Replace the battery if it seems to be running out of power sooner than it should. Please use the original lithium battery.

- Turn the motor handpiece power off.
- Use tweezers etc. to open the rubber cover and then remove the screw.
- Remove the battery cover.
- Remove the old battery and disconnect the connector.
- Connect the new battery and put it in the motor handpiece.
- Replace the cover and its screw.

It is recommended to contact local distributors or manufacturer to replace the battery.

4.12. Oiling of contra angle

Only the original oil injection nozzle can be used for oiling of contra angle. The contra angle needs to be lubricated after cleaning and disinfection, but before sterilization.


- Firstly, screw the injecting nozzle into jet of oil bottle (around 1 to 3 circles).
- Next, plug the nozzle into the end part of contra angle, and then grease the contra angle for 2-3s till the oil flow out of contra angle head part.
- Vertically place the end part of contra angle more than 30 minutes to let go the redundant oil under gravity.

Warnings

Motor handpiece cannot be filled with oil.

- To avoid the contra angle from flying out for the pressure, use hand to safely hold the contra angle while greasing.

- Do not use a swirling nozzle. Swing nozzle can only be used for injection of gas, not for oiling.

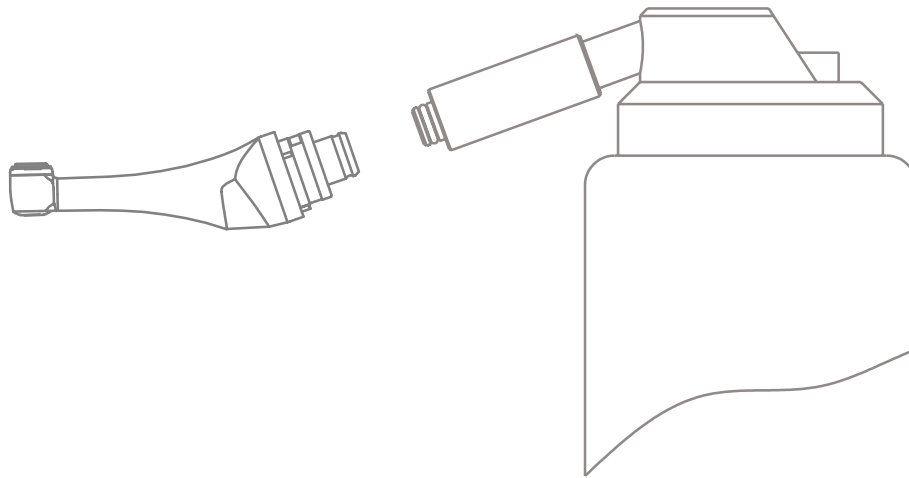

5. Troubleshooting

Failure	Possible cause	Solutions
The motor handpiece does not rotate	Chose EAL mode, EAL mode is only for canal measurement	Changing to CW, CCW, F-R or ATR mode
There is continuous beep sounds after starting the motor handpiece	The continuous beep sound is indicating that the motor handpiece is under CCW mode	Stop the motor handpiece and change the operating mode to CW Mode
Contra angle calibration failure	Calibration failure caused by strong resistance of contra angle	Clean the contra angle, and recalibrate after oil injection
The time of endurance becomes shorter after charging	Battery capacity becomes smaller	Please contact local distributor or manufacturer
No sound	Beeper Volume set to 0. Vol.0: Mute	Set Beeper Volume to 1, 2, 3
The continuously rotating file is stuck at the root canal	Incorrect specification setting. Too high load torque of file	Choose CCW Mode, start the motor handpiece, and take the file out

6. Reprocessing process

6.1. Foreword

For hygiene and sanitary safety purposes, the Motor Handpiece, AC adapter and Charging base must be cleaned and disinfected, the contra-angle, the lip hook, the file clip, the protective silicon cover and the touch probe must be cleaned, disinfected and sterilized before each usage to prevent any contamination. This concerns the first use, as well as all subsequent uses.

6.2. General recommendations

- Use only a disinfecting solution which is approved for its efficacy (VAH/DGHM-listing, CE marking, FDA and Health Canada approval) and in accordance with the DFU of the disinfecting solution manufacturer.
- Do not place the contra-angle in a disinfectant solution or in an ultrasonic bath.
- Do not use chloride detergent materials.
- Do not use bleach or chloride disinfectant materials.
- For your own safety, please wear personal protective equipment (gloves, glasses, mask).
- The user is responsible for the sterility of the product for the first cycle and each further usage as well as for the usage of damaged or dirty instruments where applicable after sterility.
- The water quality has to be convenient to the local regulations especially for the last rinsing step or with a washer-disinfector.
- To sterilize the endodontic files, refer to the manufacturer's instructions for use.

The contra-angle needs to be lubricated after cleaning and disinfection, but before sterilization.

6.3. Cleaning and disinfection steps for the motor handpiece, the AC adapter and the base.

Before and After each use, all the objects that were in contact with infectious agents should be cleaned using towels impregnated with a disinfecting and detergent solution (a bactericidal, fungicidal and aldehyde free solution) approved by VAH/DGHM-listing, CE marking, FDA and Health Canada.

Warnings

Do not sterilize the motor handpiece, the AC adapter and the base.

▪ Pre-Op processing

Before each use, the handpiece, AC adapter and base must be cleaned and disinfected. The specific steps are as follows:

Warnings

The handpiece, AC adapter and base cannot be cleaned and disinfected with automatic equipment. Manual cleaning and disinfection is required.

▪ Manual cleaning steps:

- Take out the handpiece, charger, and base on the workbench.
- Wet the soft cloth completely with distilled water or deionized water, and then wipe all the surfaces of the components such as the handpiece, charger, base, etc. until the surface of the component is not stained.
- Wipe the surface of the component with a dry soft nap-free cloth.
- Repeat the above steps at least 3 times.

Note: use distilled water or deionized water for cleaning at room temperature.

▪ Manual disinfection steps:

- Soak the dry soft cloth with 75% alcohol.
- Wipe all surfaces of headpiece, charger, base and other components with a wet soft cloth for at least 3 minutes.
- Note: the cleaning and disinfection must be performed within 10min before use. The disinfectant used must be used immediately, no foaming is allowed. In addition to 75% alcohol, you can use non-residue disinfectants such as Oxytech from Germany, but you must respect the concentration, temperature and time specified by the disinfectant manufacturer. After cleaning and disinfecting the handpiece, you must install a disposable isolation sleeve before use and repeat steps 1, 2 and 3 to clean the disposable isolation sleeve (for detailed installation steps, see section 2.7).

- Post-Op processing

After each use, clean and disinfect the handpiece, charger, and base within 30 minutes. The specific steps are as follows:

- Tools: nap-free soft cloth, tray.
- Remove the contra-angle from the handpiece, place it in a clean tray, and then remove the disposable isolation sleeve from the handpiece.
- Soak the nap-free soft cloth with distilled water or deionized water, and then wipe all the surfaces of the components such as the handpiece, charger, base, etc. until the surface of the component is not stained.
- Wet the dry soft cloth with 75% alcohol, and then wipe all surfaces of the handpiece, charger, base and other components for 3 minutes.
- Put the handpiece, charger, base and other components back into the clean storage area. Note: the cleaning and disinfection must be performed within 10min before use. The disinfectant used must be used immediately, no foaming is allowed. In addition to 75% alcohol, you can use non-residue disinfectants such as Oxytech from Germany, but you must respect the concentration, temperature and time specified by the disinfectant manufacturer.

6.4. The cleaning, disinfection and sterilization of contra-angle, lip hook, file clip, protective silicon cover, touch probe are as follow.

Unless otherwise stated, they will be hereinafter referred to as "products".

Warnings

The use of strong detergent and disinfectant (alkaline pH>9 or acid pH<5) will reduce the life span of products. And in such cases, the manufacturer takes no responsibility.

- Resistance to sterilizing procedure

The products have been designed for a large number of sterilization cycles. The materials used in manufacture were selected accordingly. However with every renewed preparation for use, thermal and chemical stresses will result in ageing of the products. The maximum number of sterilizations

for products is 250 times.

- Preparation at the Point of Use

The post-operative process must be carried out immediately, no later than 30 minutes after the completion of the operation. The steps are as follows:

- Remove the shanks/files and disconnect the contra-angle handpiece from Motor handpiece.
- Remove gross soiling of the instrument with cold water (<40°C) immediately after use. Don't use hot water (>40°C) as this can cause the fixation of residuals which may influence the result of the reprocessing process.


- Transportation

The products should be safely stored and transported to the point of reprocessing to avoid any damage and environment pollution.


- Preparation for reprocessing

The products must be reprocessed in a disassembled state:


- Press the push-button and pull out the shank/file.

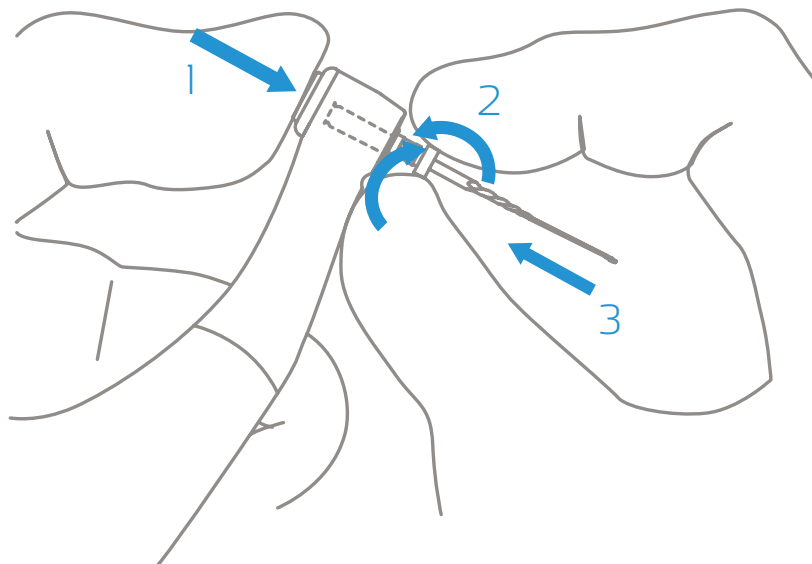

- When removing the protective silicon cover, pull it straight out slowly.

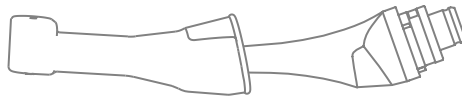

- When inserting and removing the contra-angle, turn the handpiece power off beforehand.

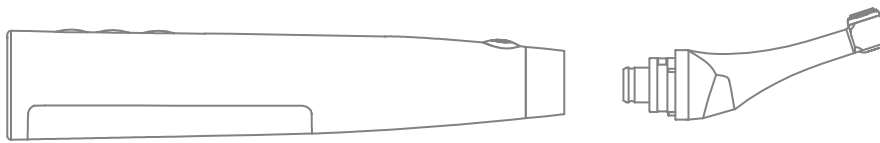

- Pre-cleaning

- Tools: tray, soft brush, clean and dry soft cloth.
- Perform manual pre-cleaning until the handpiece is visually clean. Rinse the chuck of bur with running water for at least 10 seconds. Clean the surface with a soft bristle brush. Note: the water temperature should not exceed 40°C during the washing stage, otherwise the protein will solidify and be difficult to remove.

- Cleaning

Regarding cleaning/disinfection, rinsing and drying, it is to distinguish between manual and automated reprocessing methods. Preference is to be given to automated reprocessing methods, especially due to the better standardizing potential and industrial safety.

- Automatic cleaning

The washer-disinfector should meet the requirements of the ISO 15883. Place the products in the washer-disinfector carefully. Ensure that products can not move freely in the washer-disinfector. The contra-angle handpiece are not permitted to contact with each other. Start the program:

- 4min pre-washing with cold water(<40°C)
- Emptying

- 5 min washing with a mild alkaline cleaner at 55°C
- Emptying
- 3 min neutralizing with warm water (>40°C)
- Emptying
- 5 min intermediate rinsing with warm water (>40°C)
- Emptying

Drying the device at 80°C for 15min. The automated cleaning processes have been validated by using 0.5% neodisher MediClean forte (Dr. Weigert)

- Disinfection

Automated Thermal Disinfection in washer/disinfector under consideration of national requirements in regards to AO value (see EN 15883). A disinfection cycle of 5 min disinfection at 93°C has been validated for the device to achieve an AO value of 3000.

- Drying

Drying of outside of instrument through drying cycle of washer-disinfector. If necessary, additional manual drying can be performed through lint free towel. Insufflate cavities of instruments by using sterile compressed air. If your washer-disinfector does not have an automatic drying function, please dry the device after cleaning and disinfection. The drying method as below:

- Spread a clean white paper (white cloth) on the flat table, place the products on the white paper (white cloth), and then dry the contra-angle with filtered dry compressed air (maximum pressure 3 bar). When no liquid is sprayed on the white paper (white cloth), it indicates that the products is completely dry.
- The device can also be dried directly in a medical drying cabinet (or oven). The recommended drying temperature is 80°C and the time should be 15minutes. Note: dry the products repeatedly if necessary (refer to section "Drying"). The air used for drying must be filtered by HEPA. The device should be dried in a clean area.


- Maintenance

- Functional Test and Visual inspection

Visually inspect the cleanliness of the Handpiece. Perform Functional test according to instructions of use. If there is still visible stain on the device after cleaning, the entire cleaning process must

be repeated. Before packaging and sterilization, make sure that the contra-angle handpiece has been maintained acc. to manufacturer's instruction. If the device is obviously damaged, smashed, detached, corroded or bent, it must be discarded and not allowed to continue to be used. If the accessories are found to be damaged, please replace it before use. And the new accessories for replacement must be cleaned, disinfected and dried.

- Use lubricant to lubricate the handpiece and dried it prior to sterilization. Aim the nozzle of lubricant bottle to the air hole at the end of the contra-angle handpiece to inject oil for 1-2 seconds.

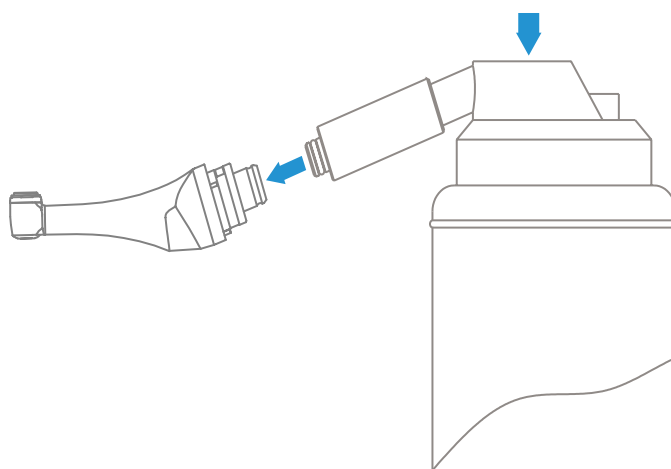

▪ Packaging

The products should be quickly packaged in a medical sterilization bag (or special holder, sterile box).

Precautions:

- Only use a legally marketed cleared sterilization pouch
- The package should withstand high temperature of 137 °C and has sufficient steam permeability
- The packaging environment and related tools must be cleaned regularly to ensure cleanliness and prevent the introduction of contaminants
- Avoid contact with different metals when packaging

▪ Sterilization

Sterilization of instruments by applying a fractionated pre-vacuum steam sterilization process (according to EN 285/EN 13060/EN ISO 17665) under consideration of the respective country requirements. Minimum requirements: at least 4 min at 132°C/134 °C (in EU: 5 min at 134 °C,

in US: 4 min at 132 °C). Flash sterilization is not allowed on lumen instruments!

- Storage

Sterilized devices should be stored in a dry, clean and dust-free environment, refer to label and instructions for use.

7. Storage, maintenance and transportation

7.1. Storage

- This equipment should be stored in a room where the relative humidity is 10% ~ 93%, atmospheric pressure is 70kPa to 106kPa, and the temperature is -20°C ~ +55°C.
- Avoid the storage in a too hot condition. High temperature will shorten the life of electronic components, damage battery, reshape or melt some plastic.
- Avoid the storage in a too cold condition. Otherwise, when the temperature of the equipment increases to a normal level, there will be dew that will possibly damage PCB board.

7.2. Maintenance

- This device do not include accessories for repair usage, the repair should be carried out by authorized person or authorized after service center.
- Keep the equipment in a dry storage condition.
- Do not throw, beat or shock the equipment.
- Do not smear the equipment with pigments.
- Calibration is recommended when using a new/other contra angle or after an extend period of operation, as the running properties can change with usage, cleaning and sterilization.
- Replace the battery if it seems to be running out of power sooner than it should.

7.3. Transportation

- Excessive impact and shake should be prevented in transportation. Lay it carefully and lightly and don't invert it.
- Don't put it together with dangerous goods during transportation.
- Avoid solarization and getting wet in rain and snow during transportation.

8. Environmental protection

Please dispose according to the local laws.














9. After service

From the date this equipment has been sold, based on the warranty card, we will repair this equipment free of charge if there are quality problems. Please refer to the warranty card for the warranty period.

10. European authorized representative

EC REP MedNet EC-REP C llb GmbH
Borkstrasse 10 · 48163 Muenster · Germany

11. Symbol instruction

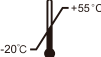	Follow Instructions for Use		Serial number
	Date of manufacture		Manufacturer
	Type B applied part		Class II equipment
IPX0	Ordinary equipment		Recovery
	Used indoor only		Keep dry
	Handle with care		Appliance compliance WEEE directive
	Humidity limitation		Temperature limitation



Atmospheric pressure for storage



CE marked product



Authorised Representa- tive
in the EUROPEAN COMMUNITY



Medical Device



Importer



Unique device identifier



Catalogue number



Sterilize in a steam
autoclave at 134°C

12. Statement

All rights of modifying the product are reserved to the manufacturer without further notice. The pictures are only for reference. The final interpretation rights belong to GUILIN WOODPECKER MEDICAL INSTRUMENT CO., LTD. The industrial design, inner structure, etc, have claimed for several patents by WOODPECKER, any copy or fake product must undertake legal responsibilities.

13. EMC-Declaration of conformity

The device has been tested and homologated in accordance with EN 60601- 1-2 for EMC. This does not guarantee in any way that this device will not be effected by electromagnetic interference. Avoid using the device in high electromagnetic environment. Technical Description Concerning Electromagnetic Emission.


Technical Description Concerning Electromagnetic Emission. Table I: Declaration - electromagnetic emissions

Guidance and manufacturer's declaration - electromagnetic emissions		
The model Endostar Performer is intended for use in the electromagnetic environment specified below. The customer or the user of the model Endostar Performer should assure that it is used in such an environment.		
Emissions test	Compliance	Electromagnetic environment - guidance
RF emissions CISPR 11	Group 1	The model Endostar Performer uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment
RF emissions CISPR11	Class B	The model Endostar Performer is suitable for used in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations /flicker emissions IEC 61000-3-3	Complies	

Guidance & Declaration — electromagnetic immunity			
The model Endostar Performer is intended for use in the electromagnetic environment specified below. The customer or the user of the model Endostar Performer should assure that It is used in such an environment			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Electrostatic discharge (ESD) IEC 61000-4-2	$\pm 8\text{kV}$ contact $\pm 2, \pm 4, \pm 8, \pm 15\text{kV}$ air	$\pm 8\text{kV}$ contact $\pm 2, \pm 4, \pm 8, \pm 15\text{kV}$ air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%
Electrical fast transient/burst IEC 61000-4-4	$\pm 2\text{kV}$ for power supply lines $\pm 1\text{kV}$ for Input/output lines	$\pm 2\text{kV}$ for power supply lines	Mains power quality should be that of a typical commercial or hospital environment
Surge IEC 61000-4-5	$\pm 0.5, \pm 1\text{kV}$ line to line $\pm 0.5, \pm 1, \pm 2\text{kV}$ line to earth	$\pm 0.5, \pm 1\text{kV}$ line to line	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	$< 5\%$ UT ($> 95\%$ dip in UT.) for 0,5 cycle $< 5\%$ UT ($> 95\%$ dip in UT.) for 1 cycle 70% UT (30% dip in UT) for 25 cycles $< 5\%$ UT ($> 95\%$ dip in UT) for 250 cycles	$< 5\%$ UT ($> 95\%$ dip in UT.) for 0,5 cycle $< 5\%$ UT ($> 95\%$ dip in UT.) for 1 cycle 70% UT (30% dip in UT) for 25 cycles $< 5\%$ UT ($> 95\%$ dip in UT) for 250 cycles	Mains power quality should be that of a typical commercial or hospital environment. If the user of the models Endostar Performer requires continued operation during power mains interruptions, it is recommended that the models Endostar Performer be powered from an uninterruptible power supply or a battery.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	30A/m	30A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

NOTE: UT is the a.c. mains voltage prior to application of the test level.

Table 3: Guidance & Declaration - electromagnetic immunity concerning Conducted RF & Radiated RF

Guidance & Declaration - Electromagnetic immunity			
The model Endostar Performer is intended for use in the electromagnetic environment specified below. The customer or the user of the models Endostar Performer should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Conducted RF IEC 61000-4-6 Conducted RF IEC 61000-4-6 Radiated RF IEC 61000-4-3	3 Vrms 150 kHz to 80 MHz 6 Vrms ISM frequency band 3 V/m 80 MHz to 2,7 GHz	3V 6V 3V/m	<p>Portable and mobile RF communications equipment should be used no closer to any part of the models Endostar Performer, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance:</p> $d = 1,2 \times P^{1/2}$ $d = 2 \times P^{1/2}$ $d = 1,2 \times P^{1/2} \text{ 80 MHz to 800 MHz}$ $d = 2,3 \times P^{1/2} \text{ 800 MHz to 2,7 GHz}$ <p>where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and "d" is the recommended separation distance in meters (m).</p> <p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, a) should be less than the compliance level in each frequency range b) Interference may occur in the vicinity of equipment marked with the following symbol:</p> 

Note: At 80 MHz end 800 MHz. the higher frequency range applies.

Note: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

- Field strengths from fixed transmitters, such as base stations for radio (cellular/ cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the model Endostar Performer is used exceeds the applicable RF compliance level above, the model Endostar Performer should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the model Endostar Performer.
- Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3V/m.

Table 4: Recommended separation distances between portable and mobile RF communications equipment and the model Endostar Performer.

Recommended separation distances between portable and mobile RF communications equipment and the model Endostar Performer			
The model Endostar Performer is intended for use in electromagnetic environment in which radiated RF disturbances is controlled. The customer or the user of the model Endostar Performer can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the model Endostar Performer as recommended below, according to the maximum output power of the communications equipment.			
Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter m		
	150kHz to 80MHz $d=1,2 \times P^{1/2}$	80MHz to 800MHz $d=1,2 \times P^{1/2}$	800MHz to 2,7GHz $d=2,3 \times P^{1/2}$
0,01	0,12	0,12	0,23
0,1	0,38	0,38	0,73
1	1,2	1,2	2,3
10	3,8	3,8	7,3
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) accordable to the transmitter manufacturer.

Note: At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

Note: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.



Poldent Sp. z o.o.
Dzika 2, 00-194, Warszawa, Poland
e-mail: poldent@poldent.pl
e-mail: endostar@endostar.eu
www.poldent.pl
www.endostar.eu




Guilin Woodpecker Medical Instrument Co., Ltd.
Information Industrial Park, Guilin National High-Tech
Zone, Guilin, Guangxi, 541004 P. R. China
Sales Dept.: +86-773-5873196/2350599
After-sales Service Dept.: +86-0773-5827898
E-mail: woodpecker4@glwoodpecker.com
Website: <http://www.glwoodpecker.com>

EC REP MedNet EC-REP C IIb GmbH
Borkstrasse 10 · 48163 Muenster · Germany

Karta gwarancyjna Warranty Card


Nazwa klienta <i>Name of Customer</i>		(I) Dla dystrybutora <i>For Distributor</i>
Adres Szczegóły <i>Address Details</i>		
Kod pocztowy <i>Postal Code</i>		
Telefon <i>Tel</i>		
Model		
Nr rękojści <i>Motor Handpiece NO.</i>		
Nr kątnicy <i>Contra-angle NO.</i>		
Data zakupu <i>Purchase Date</i>		
Osoba kontaktowa <i>Contact Person</i>		
Data <i>Date</i>	Rejestr konserwacji <i>Maintenance Record</i>	Serwisant <i>Repairer</i>

 **Guilin Woodpecker Medical Instrument Co.,Ltd.**
 Information Industrial Park, Guilin National High-Tech
 Zone, Guilin, Guangxi, 541004 P. R. China
 Sales Dept.: +86-773-5873196/2350599
 After-sales Service Dept.: +86-0773-5827898
 E-mail: woodpecker4@glwoodpecker.com
 Website: http://www.glwoodpecker.com

Dystrybutor/ <i>Distributor</i>
Pieczętka/ <i>Seal</i>

Karta gwarancyjna Warranty Card

Nazwa klienta <i>Name of Customer</i>		(II) Powrót do produkcji <i>Return to Manufacturer</i>
Adres Szczegóły <i>Address Details</i>		
Kod pocztowy <i>Postal Code</i>		
Telefon <i>Tel</i>		
Model		
Nr rękojści <i>Motor Handpiece NO.</i>		
Nr kątnicy <i>Contra-angle NO.</i>		
Data zakupu <i>Purchase Date</i>		
Osoba kontaktowa <i>Contact Person</i>		
Data <i>Date</i>	Rejestr konserwacji <i>Maintenance Record</i>	Serwisant <i>Repairer</i>

 **Guilin Woodpecker Medical Instrument Co.,Ltd.**
 Information Industrial Park, Guilin National High-Tech
 Zone, Guilin, Guangxi, 541004 P. R. China
 Sales Dept.: +86-773-5873196/2350599
 After-sales Service Dept.: +86-0773-5827898
 E-mail: woodpecker4@glwoodpecker.com
 Website: http://www.glwoodpecker.com

Dystrybutor/ <i>Distributor</i>
Pieczętka/ <i>Seal</i>

Warunki gwarancji
Warranty Card

I Okres gwarancji:

Okres gwarancji liczony od daty zakupu przez
dentystę.

Baza ładująca, rękojeść, zasilacz sieciowy	2 lata
Kątnica	1 rok
Inne części zamienne	6 miesięcy

I Period validity:

*The base, handpiece, power adapter have two years
warranty period from the date of purchase. The
contra-angle has one year warranty period. Other
spare parts have six months warranty period.*

II Zakres gwarancji:

W okresie ważności gwarancji jesteśmy
odpowiedzialni za wszelkie usterki spowodowane
przez problemy jakościowe lub technikę i budowę
produktów.

II Range of warranty:

*Within the warranty period of validity, we are
responsible for any troubles caused by quality
problems or products technique and structure.*

**III Następujące przypadki są wyłączone z naszej
gwarancji:**

1. Uszkodzenia spowodowane nieprzestrzeganiem
instrukcji obsługi lub brakiem spełnienia
wymaganych warunków.
2. Uszkodzenia produktu spowodowane niewłaściwą
obsługą lub demontażem bez upoważnienia.
3. Uszkodzenia spowodowane przypadkowym
upuszczeniem lub uderzeniem produktu przez
użytkowników.
4. Uszkodzenia spowodowane niewłaściwym
transportem lub przechowywaniem.
5. Brak pieczętki dystrybutora lub niekompletne
wypełnienie karty gwarancyjnej.

III The following are beyond our warranty:

1. The damage caused by disobeying the operation
instruction or lack of the needed condition.
2. The damage caused by unsuitable operation or
disassembly without authorization.
3. The damage on product that caused by users'
unexpected drop or impact to product.
4. The damage caused by inadvisable transportation
or preservation.
5. There isn't the seal of distributor or the warranty
card isn't filled in completed.

Warunki gwarancji
Warranty Card

I Okres gwarancji:

Okres gwarancji liczony od daty zakupu przez
dentystę.

Baza ładująca, rękojeść, zasilacz sieciowy	2 lata
Kątnica	1 rok
Inne części zamienne	6 miesięcy

I Period validity:

*The base, handpiece, power adapter have two years
warranty period from the date of purchase. The
contra-angle has one year warranty period. Other
spare parts have six months warranty period.*

II Zakres gwarancji:

W okresie ważności gwarancji jesteśmy
odpowiedzialni za wszelkie usterki spowodowane
przez problemy jakościowe lub technikę i budowę
produktów.

II Range of warranty:

*Within the warranty period of validity, we are
responsible for any troubles caused by quality
problems or products technique and structure.*

**III Następujące przypadki są wyłączone z naszej
gwarancji:**

1. Uszkodzenia spowodowane nieprzestrzeganiem
instrukcji obsługi lub brakiem spełnienia
wymaganych warunków.
2. Uszkodzenia produktu spowodowane niewłaściwą
obsługą lub demontażem bez upoważnienia.
3. Uszkodzenia spowodowane przypadkowym
upuszczeniem lub uderzeniem produktu przez
użytkowników.
4. Uszkodzenia spowodowane niewłaściwym
transportem lub przechowywaniem.
5. Brak pieczętki dystrybutora lub niekompletne
wypełnienie karty gwarancyjnej.

III The following are beyond our warranty:

1. The damage caused by disobeying the operation
instruction or lack of the needed condition.
2. The damage caused by unsuitable operation or
disassembly without authorization.
3. The damage on product that caused by users'
unexpected drop or impact to product.
4. The damage caused by inadvisable transportation
or preservation.
5. There isn't the seal of distributor or the warranty
card isn't filled in completed.