Flexible root canal irrigation needles**BRIEF OF THE PRODUCT**

Endostar ENDOflex needles are flexible cannulas with radial openings for liquid ejection along the canal. They are used when connected to a Luer lock syringe to remove debris from the root canal system during root canal treatment.

WARNINGS

- 1) Only skilled dentists are allowed to use.
- 2) Do not use this product except for the dental service and treatment. Use it in accordance with the intended use.
- 3) This product must be disinfected before use.
- 4) Do not use ultrasonic equipment cleaning automated

DIRECTION FOR USE**Preparation**

- 1) Open the packaging.
- 2) Take out the needle and disinfecting as required.
- 3) Prefill the syringe with one of the compatible irrigation solutions (stabilized sodium hypochlorite up to 6%, EDTA 17%, Chlorhexidine digluconate 2%, Acid citric up to 40%, Dual Rinse® HEDP). Make sure the irrigation solution has been properly stored, avoid the formation of any crystal which may block the small hole at the top of the needle.
- 4) Insert the Connector Part of the needle into the the previously filled Luer-Lock syringe and lock the needle by rotating it clockwise.
- 5) Ensure the needle is firmly locked on the syringe and expel any residual air from the syringe.
- 6) Identify the 4 depth marks on the cannula (18 mm, 19 mm, 20 mm and 22 mm). Can use a stopper to define the working length.
- 7) Try and check the pressure needed to create two equal jets exiting the needle with property speed.

Irrigation

- 8) Start irrigating in the pulp chamber and gently insert the cannula into the canal orifice. Can pre-bend the tip to facilitate access into posterior teeth.
- 9) Use a scouting file to create a smooth canal pathway.
- 10) Determine the working length and re-irrigate with 1 or 2 ml.
- 11) As a rule, after each trimming for the root canal in the sequence, irrigate with 1 or 2 ml of solution. Make sure that irrigation solution have gap to flowback. Check the canal have been irrigated clean.
- 12) At the final rinse, recommended to flush the canal with EDTA followed by sodium hypochlorite.
- 13) After use, disconnect the cannula and dispose according to local regulations.

PRECAUTIONS FOR USE

Check the length of the canal to be irrigated and use the 18-, 19-, 20- and 22-mm length markings to ensure proper positioning of the needle.

CLEANING AND DISINFECTION

- **Preparation:** Wear clean gloves prior to handling and disinfecting.
 - **Cleaning:** Perform a large amount of flushing (at least 1 minute) under flowing
 - **Disinfection:** The recommended disinfectant solution is 5.25% Sodium Hypochlorite solution (bleach) and wiping with 70% Isopropyl Alcohol (IPA).
- Disinfect the Root Canal Irrigation Needle by fully immersing in a bath of 5.25% Sodium Hypochlorite solution for one minute. Gently wipe the Root Canal Irrigation Needle with a sterile gauze moistened by 70% Isopropyl Alcohol

DRYING

Air dry the Root Canal Irrigation Needle.

INSPECTION

Visually inspect the device for any damage prior processing. If the device is fractured, cracked, etc., discard and do not use.

STERILIZATION

Sterilization may lead to damage of the equipment.

STORAGES

- 1) Avoid storing at high temperature, humidity and direct sunlight. Keep liquids away. Store it at room temperature.
- 2) Do not damage or make a pinhole to packaging materials.
- 3) The product is subject to be improved without previous notice. Enforce the first-in first-out method for stock management.
- 4) To prevent contamination, store processed device in covered storage and in a drawer or cabinet until use.

5) Do not keep under germicidal lamp to avoid deterioration.

6) The shelf time Root Canal Irrigation Needle are 5 years.

TRANSPORTATION

1) Packaged products can be transported using standard transportation methods or as specified in the specified in the contract. It strictly prohibited to mix with harmful, inflammable and explosive material. Avoid heavy pressure, exposure to sunlight, and direct contact with rain and snow during transportation.

2) When the package was broken, then it may be damaged or polluted, must be discard and do not use.

DISPOSAL

For proper disposal, always observe national laws and recommendations of the authorities.

EXPLANATION OF RELATED SYMBOLS

	Medical device		
	For Dental Use Only		
	Do not reuse		
	Authorized representative in the European Union		
	Batch code		
	Non-Sterile		Unique Device Identifier
	Expiration date		Date of manufacture
	Consult instructions for use		Fragile, handle with care
	Manufacturer		Recyclable packaging
	CE marking		This way up
	Importer		Keep away from sunlight
	Catalogue number		Keep dry

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**Shenzhen Rogin Medical Co., Ltd**